



bio-bottle **Packing Instructions:**

1. Remove the bio-bottle from the Fibreboard Outer Carton
2. Remove the lid from the bio-bottle
3. Remove the Bubble Bag and the Twin Diagnostic Label from the bio-bottle
4. Wrap the sample(s) inside the Bubble Bag
5. Place the bubble bag inside the bio-bottle
6. Screw the lid of the bio-bottle on firmly
7. Place the bio-bottle inside the Fibreboard Outer Carton and close
8. Seal the closure with packing tape
9. Place the correct Label on the outside of the Fiberboard Outer Carton
10. Ensure that all markings are compliant to the required regulations.

Note, NEVER place Dry Ice inside bio-bottle

bio-bottle **Package Contents:**

1. Fiberboard Outer Carton
2. bio-bottle
3. Twin Diagnostic Label (Biological or Infectious Substance)
4. Bubble Bag
5. Absorbent Material